# DD Implant Screw

**DE** Gebrauchsanweisung, **EN** Instructions for use, **FR** Notice d'utilisation, **IT** Istruzioni per l'uso, **ES** Instrucciones de uso, **NL** Gebruiksaanwijzing, **EL** Οδηγίες χρήσης για το, **PT** Instruções de utilização, **TR** Kullanım talimatı, **RU** Инструкция по применению, **SV** Bruksanvisning, **ET** Kasutusjuhend, **DA** Brugsanvisning, **NO** Bruksanvisning, **FI** Käyttöohje, **IS** Notkunarleiðbeiningar, **PL** Instrukcja używania, **CS** Návod k použití skenovacího tělíska, **HU** Használati útmutató, **RO** Instrucțiuni de utilizare, **SK** Návod na použitie, **BG** Инструкция за употреба, **SR** Упутство за употребу, **FA** راهنمای استفاده از, **AR** دليل استخدام, **ZH** 使用说明书, **KO** 사용 설명서, **HR** Upute za uporabu, **GA** Treoracha úsáide, **LV** lietošanas pamācība, **LT** Naudojimo instrukcija, **SL** Navodila za uporabo, **MT** Struzzjonijiet għall-użu, **JA** 取扱説明書, **BS** Uputstvo za upotrebu

<b>DE</b>	DEUTSCH	Gebrauchsanweisung	4
			5
<b>EN</b>	ENGLISH	Instructions for use	6
			7
<b>FR</b>	FRANÇAIS	Notice d'utilisation	8
			9
<b>IT</b>	ITALIANO	Istruzioni per l'uso	10
			11
<b>ES</b>	ESPAÑOL	Instrucciones de uso	12
			13
<b>NL</b>	NEDERLANDS	Gebruiksaanwijzing	14
			15
<b>EL</b>	ΕΛΛΗΝΙΚΑ	Οδηγίες χρήσης για το	16
			17
<b>PT</b>	PORTUGUÊS	Instruções de utilização	18
			19
<b>TR</b>	TÜRKÇE	Kullanım talimatı	20
			21
<b>RU</b>	РУССКИЙ	Инструкция по применению	22
			23
<b>SV</b>	SVENSK	Bruksanvisning	24
			25
<b>ET</b>	EESTI	Kasutusjuhend	26
			27
<b>DA</b>	DANSK	Brugsanvisning	28
			29
<b>NO</b>	NORSK	Bruksanvisning	30
			31
<b>FI</b>	SUOMI	Käyttöohje	32
			33
<b>IS</b>	ÍSLENSKA	Notkunarleiðbeiningar	34
			35
<b>PL</b>	POLSKI	Instrukcja używania	36
			37
<b>CS</b>	ČEŠTINA	Návod k použití skenovacího tělíska	38
			39

<b>HU</b>	MAGYA	Használati útmutató	40
			41
<b>RO</b>	ROMÂNĂ	Instrucțiuni de utilizare	42
			43
<b>SK</b>	SLOVENSKY	Návod na použitie	44
			45
<b>BG</b>	БЪЛГАРСКИ	Инструкция за употреба	46
			47
<b>SR</b>	СРПСКИ	Упутство за употребу	48
			49
<b>FA</b>	فارسی	راهنمای استفاده از	50
			51
<b>AR</b>	عربي	دليل استخدام	52
			53
<b>ZH</b>	中文	使用说明书	54
			55
<b>KO</b>	한국어	사용 설명서	56
			57
<b>HR</b>	HRVATSKI	Upute za uporabu	58
			59
<b>GA</b>	GAEILGE	Treoracha úsáide	60
			61
<b>LV</b>	LATVIEŠU VALODA	lietošanas pamācība	62
			63
<b>LT</b>	LIETUVIŲ KALBA	Naudojimo instrukcija	64
			65
<b>SL</b>	SLOVENŠČINA	Navodila za uporabo	66
			67
<b>MT</b>	MALTI	Struzzjonijiet għall-użu	68
			69
<b>JA</b>	日本語	取扱説明書	70
			71
<b>BS</b>	BOSANSKI	Uputstvo za upotrebu	72
			73

**Inhalt:**

1 x DD Implant Screw, silberfarben

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1 x DD Implant Screw, silver

**Product description:**

The Dental Direkt Implant Screw is an accessory for the Solid Connect® products and can only be used in combination with them. It is used for fixation between abutment, titanium bonding base or scan body and implant. The Implant Screw is available for different implant systems. The Dental Direkt Implant Screw is made of Ti6Al4V (medical grade 5) for single use and is supplied non-sterile.

**Intended purpose:**

DD Solid Connect® implant components are used to support prosthetic restorations in combination with endosseous dental implants in the upper and/or lower jaw.

**Indication:**

For fixation of individual abutments, titanium bonding bases or scan bodies to implants in the upper and lower jaw

**Please note:** DD Solid Connect® implant components from a particular series can only be combined with the corresponding compatible implant system. This also applies to the platform diameters.

**Contraindication:**

Intolerance to the constituents

**Please note:** Observe the contraindications specified by the original implant manufacturer used in each case.

**Intended users:**

DD Solid Connect® implant components should only be used by dentists and dental technicians in compliance with the procedures in the instructions for use.

**Intended patient group:**

Fixed restorations using Dental Direkt Solid Connect® implant components are suitable for the permanent dentition in adult patients of any gender and nationality.

**Instructions for use in the dental practice:**

The individual prosthetic restoration is placed in the patient's mouth using the silver Implant Screw.

**Caution:** The Implant Screw must be tightened to the torque specified for the system (see table).

**Please note:** It is recommended to tighten the Implant Screw 10 minutes after placement.

**Material:**

The DD Implant Screw is made of Ti6Al4V (medical grade 5) ASTM 136 ELI.

**Cleaning, disinfection and sterilization:**

The Implant Screw is cleaned at Dental Direkt, but not sterilized, before being packaged. Before being used on the patient, the Implant Screw must be cleaned, disinfected and sterilized following the procedure described by Dental Direkt.

**Cleaning/disinfection:**

Automatic cleaning in a suitable washer-disinfector with an appropriate cleaning program for medical instruments (e.g. "Vario TD Program", Miele).

OR

The following manual cleaning procedure:

- Rinse with cold tap water for 1 minute
- Rinse the screw channel using a disposable syringe (min. capacity 10 ml)
- Clean in the ultrasonic cleaner at 60°C (140°F) +/- 2°C, e.g. using Dr. Weigert neodisher® LM 2 for 10 minutes at a concentration of 5%
- Neutralize with cold distilled water for 2 minutes
- Rinse in the ultrasonic cleaner using distilled water at 60°C (140°F) for 10 minutes

**Sterilization:**

Double-seal the products in sterilization film conforming to EN ISO 11607-1. Steam sterilize in a suitable sterilizer with 3x fractionated prevacuum: 134°C (273°F) for 5 minutes, drying time: 20 minutes. Then store in sterile packaging.

**Caution:** The DD Implant Screw is suitable for single use only. Any reuse may lead to hazards due to infection.

**Possible side effects and interactions:**

Allergic reactions or skin irritations are possible in very rare cases. Different alloys in the oral cavity can lead to galvanic reactions when in direct contact.

**Disposal:**

In compliance with local regulations.

**Warranty:**

Our products undergo strict quality control and conform to the latest technical standards. Please refer to our current warranty terms for further details.

**Please note:** Any serious incident that occurs in connection with the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is located

**Please note:** The safety and clinical performance summary report can be requested at [info@dentaldirekt.de](mailto:info@dentaldirekt.de).

We are continuously developing and enhancing our products, and therefore reserve the right to make changes. The latest version of the instructions for use can also be found on our website at:

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This version replaces all previous versions.

**Explanation of symbols:**

Manufacturer

Date of manufacture

Batch code

Catalog number

Keep dry

Non-sterile

Do not use if package is damaged

Torque

Consult electronic instructions for use  
[www.dentaldirekt.de/IFU](http://www.dentaldirekt.de/IFU)

Caution: Under U.S. federal law, the device may only be sold by or on behalf of a dentist.

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