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Instruction Manual

DDcamWAX

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Material-related characteristics

Composition: synthetic wax

Specifications

Colour	grey
Odour	odourless
Drop melting point	100 – 130 °C
Flash point	not applicable
Density	0,97 g/cm ³
Solubility in water	not soluble

Explanation of symbols

Manufacturer	Consult instructions for use
Batch code	Catalogue No.
Store dry	Protect from direct sun light

Instruction Manual

- General information**
 DD cam WAX blanks can be machined with all milling systems. Only recommended milling tools for the milling system should be used for processing and milling. Please always make sure customers follow the instructions for use applicable to the product.
- Storage**
 Protect from direct sunlight. Protect from moisture.
- Areas of application**
 DD cam WAX is not intended for intraoral use. The material can be used to make frameworks for crown and bridge models. These models can then be used in the casting process, e.g. for the fabrication of dental restorations. The wax burns out without trace.
- Finishing**
 The milled frameworks can separated from the blanks with tungsten carbide cross-cut burs suitable for wax or with suitable cutting discs.
 - Spruing**
 The rules for designing and making sprues depend on the alloy or the press ceramic used. You can find detailed information on the alloys in the instructions for use of the manufacturer.
 - Investing**
 Use a ceramic ring liner. Mix the investment material under vacuum and invest in the usual way. Always follow the instructions for use pertaining to the investment material and the alloy or press ceramic being used.
- Disposal**
 DD cam WAX made from synthetic wax. Residues can be reused for conventional wax jobs. It can also be disposed of through the domestic refuse disposal system. DD cam WAX is water-insoluble, inert and presents no hazard to the ground water.